
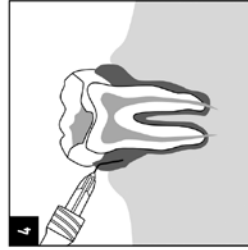
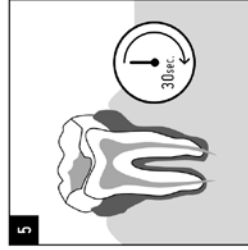
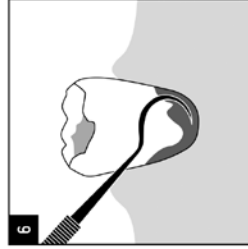
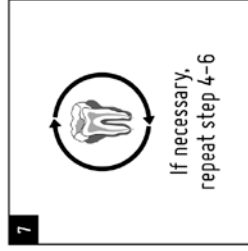
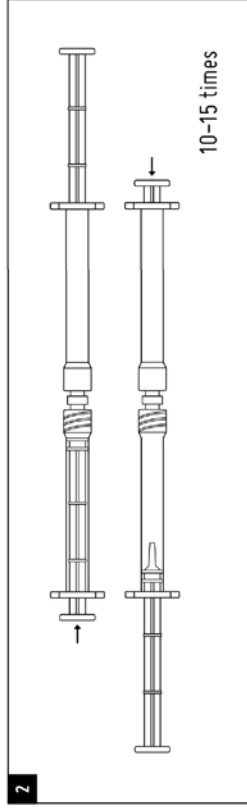
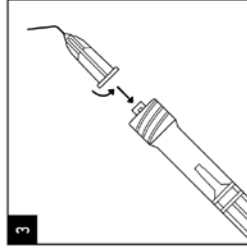
PERISOLV® INSTRUCTIONS FOR USE

LV	Lietošanas pamācība	3
ET	Kasutusjuhend	4
GB	Instructions for use	5
DE	Gebrauchsanweisungen	6
ES	Instrucciones de uso	7
PT	Instruções de utilização	8
FR	Mode d'emploi	9
IT	Istruzioni per l'uso	10
SE	Bruksanvisning	11
NO	Bruksanvisning	12
DK	Brugsanvisning	13
NL	Instructies voor gebruik	14
PL	Instrukcja użytkowania żelu	15
BG	Инструкции за употреба	16
CZ	Návod k použití	17
FI	Käyttöohje	18
HU	Használati útmutató	19

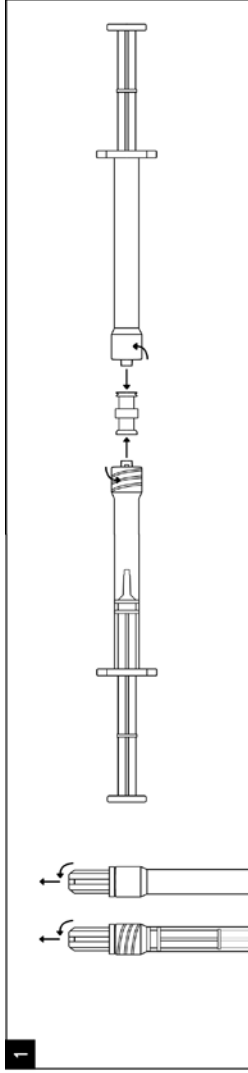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GB

PERISOLV® DENTAL DEBRIDEMENT GEL

Perisolv shall be used under the supervision of a qualified dental professional.

INTENDED USE

Perisolv is an adjunctive therapy to mechanical debridement of periodontal packets and peri-implant defects. Perisolv softens degenerated tissue and facilitates the removal of the biofilm.

PRODUCT DESCRIPTION

Perisolv consists of two solutions provided in two separate syringes with a closable cap. The solutions are mixed to form a gel at the time of use. Each package consists of two syringes, a connector for mixing and an application tip. The mixing of the solutions forms an opaque gel, with slight odour of chlorine. The mixed gel is applied to the periodontal packet with the application tip. After application scaling and / or root planing is performed to help in the mechanical removal of degenerative tissue and biofilm from the application site.

INDICATIONS FOR USE

Perisolv is indicated for patients diagnosed with periodontitis, peri-implant mucositis and peri-implantitis.

Perisolv is intended for single patient single use.

INSTRUCTIONS FOR USE

- 1 Keep the product at room temperature a short time before use. Take the syringes out of the aluminium pouch together with the connector and the application tip. Hold the syringe with the opening upwards and remove the caps. Apply the connector and mount the syringes together.
- 2 Mix the solutions by pressing alternately on the end of each syringe 10-15 times until the solutions are homogenous.

- 3 Press all the liquid into the transparent syringe. Remove the connector. Attach the application tip to the transparent syringe.
- 4 Dispense Perisolv into the periodontal packets that will be treated. Fill the packet all the way up to the gum line starting at the most apical aspect. Use product immediately after mixing.
- 5 Leave the gel in the periodontal packet for 30 seconds.
- 6 Per/arm scaling and/or root planing.
- 7 If necessary, repeat step 4-6.

INGREDIENTS

Opaque viscous solution consisting of water, carboxymethyl cellulose, sodium hypochlorite, sodium chloride, amino acids (glutamic acid, leucine, lysine) and titanium dioxide with alkaline pH.

PRECAUTIONS

- Perisolv shall not be used if the patient is suspected to be allergic to any of the ingredients.
- Perisolv shall not be used together with chlorhexidine.
- In case of contact with the eyes, rinse carefully with plenty of water for several minutes.
- Shall not be injected.

HANDLING, STORAGE AND DISPOSAL

Keep the product refrigerated (2–8° C) in its original packaging during storage. Can stain clothing.

Disposal should be handled in accordance to local regulations and instructions.

SHELF-LIFE

Use prior to expiry date as found on the package label.

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RLS Global AB
Sabiedrība „RLS Global AB”